

XOG MUHIIM AH!

Si taxaddar leh u baro SalivaPOD oo akhri dhammaan tilmaamaha isticmaalka kahor intaadan bilaabin.
Iska qaadida candhuufta waxaa si habboon loo sameeyaa subaxdii.
Waa inaadan cunin, cadayan ilkahaaga, ama aanad isticmaalin waxyaalaha laga sameeyay tubaakada 30 daqiqo kahor muunada.

App-ka SalivaCODE waxaa loogu talagallay inuu noqdo hage loo isticmaalo.

1A-I Haddii aad isticmaaleysyo

SalivaCODE App si aad ugu diiwaangeliso muunada candhuufta lambarka QR: Raac dhammaan tillaaboo yinka barnaamijka markii ugu horreysay ee aad soo qaaddo muunada candhuufta.

1A-II Haddii aad isticmaaleysid warqadda ku qoran buug yaraha

xirmada: Qalin ku buuxi magacaaga iyo nambarka aqoonsigaaga ku qaran,

SOO CELI INTA MUUNADA LA QAADIN, ka dibna ku dhaji warqadda ku qoran SalivaPOD ama tubbada baaritaanka.

1B Ka fur furka cawlka oo dhinac dhigo, ha tuurin!

Hadda waxaaad diyaar u tahay qaadida candhuufta. AKHRISO DHAMMAAN 1C INTA AADAN SI TARTIIB AH UGU SHUBBIN CANDHUUFTA WEELKA AFKA!

1C Uqabo si toos ah salivaPOD. Ku ururi candhuufta afkaaga hore oo si tartiib ah candhuufta ugu riix qeybta afka ee cad.

Ku celi dhowr jeer illaa



CANDHUUFTA DAREERAHA AHI AY KA SOO GAARAYSO Qadka (1D). La soco candhuufta dareeraha ah ee tuubada gudaha inta u dhexeysa buuxi mid kasta. Iftiinka wanaagsan wuxuu caawiyaa muuqaalka.

Iska ilaali hawo iyo xumbooyin, ha afuufin! Si qafiif ah u nuug xumbada

dusha sare ee silsiladda gudaha, ama u dhig tijaabada dhinac muddo gaaban ka hor intaadan ku celin (1E).

1F MUHIIM AH! Haddii candhuuf badani soo gasho qolka dibadda, waa in laga qaadaa si looga fogaado daadashada. Daboolka dib ugu celi afkiisa iyadoo fallaarta u jeeddo calaamadda. Si adag furka hoos ugu riix ilaa uu ka istaago (1G).

Ku samee diiwaangelinta barnaamijka app-ka ama ugu dambeyn ku dheji warqadda gacanta lagu qoray SalivaPOD (ama tuubada tijaabada, KADIB samee tallaaboo yinka bogga 2). U soo gudbi muunada AMA qabo bogga 2aad haddii aad heshay tuubo tijaabo ah, masaxaada jeermis dile ah iyo / ama bac.

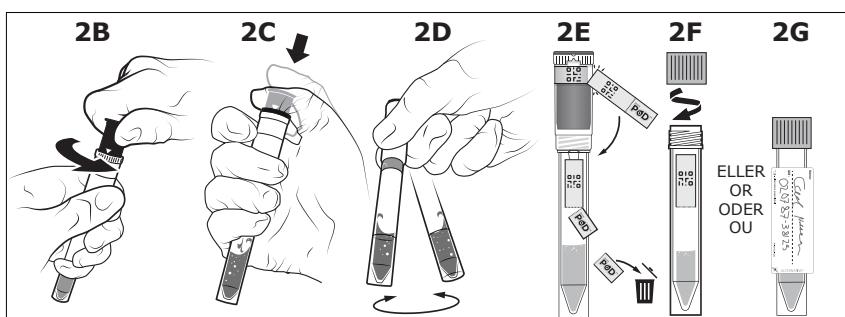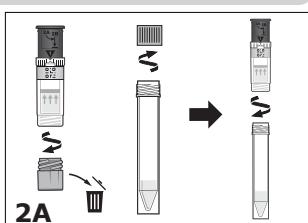

Labada salivaPOD™ iyo tuubada tijaabada ah ee ku habboon, ku samee waxyaabaha soo socda:

2A Ka fur labada koob. Furka u hay tubbada baaritaanka. Dib ayaa loo isticmaali doonaa. Isku soo ururi tubbada tijaabada ee salivaPOD™ salka hoose.

Ku xir SalivaPOD™ tuubada baaritaanka.

2B Marooji daboolka qiyaastii. 1cm saacad celcelis ahaan.

2C Furka daboolka hoos ugu riix dhanka hoose ee weelka.

2D Wareeji tuubada tijaabada si aad ugu qasto waxyaabaha ku jira.

2E Markaad isticmaaleysyo lambarka QR: Ka jar mid ka mid ah calaamadaha QR oo u gudbi tan tuubada baaritaanka Ku dhammee adigoo ka jaraya qaybta astaanta. Haddii nambarka QR halleyasan yahay, isticmaal lambarka isbeerka eh.

2F Ka fur SalivaPOD tuubada baaritaanka si adagna ugu xir daboolka guduudan tuubada baaritaanka.

2G Haddii aad isticmaaleysyo calaamada warqadda, tan waa in ay ku dhegto dhererkeeda tuubada baaritaanka jeermiska ka dib (2H).

2H Haddii aad heshay masaxaad jeermiska disha iyo/ama bac: Ku nadiifi tubbada baaritaanka mashiinka jeermiska dila KAHOR INTA AADAN QORIN CALAAMADA GACAN KU QORKA EE LAGU CALAAMEYAY (1A-II), geli tuubada calaamadeysan ee muunada bacda oo xir bacda.